


Alltest hCG Pregnancy Rapid tests


Alltest hCG Rapid Dipstick test

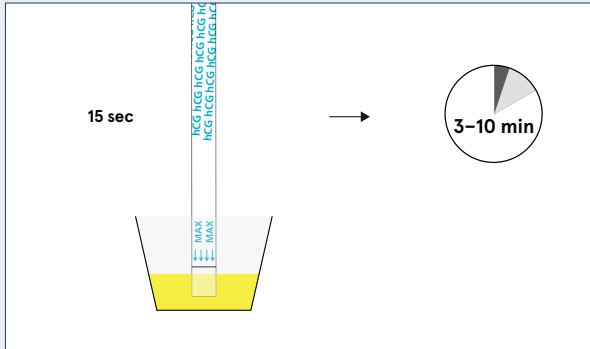

With arrows pointing down, place the dipstick into the tube. Immerse the test in the urine for at least 15 seconds. Place the dipstick on a non-absorbent flat surface and start the timer.

Read the results at 3 minutes. Do not interpret the results after 10 minutes.



Alltest hCG Rapid Cassette test

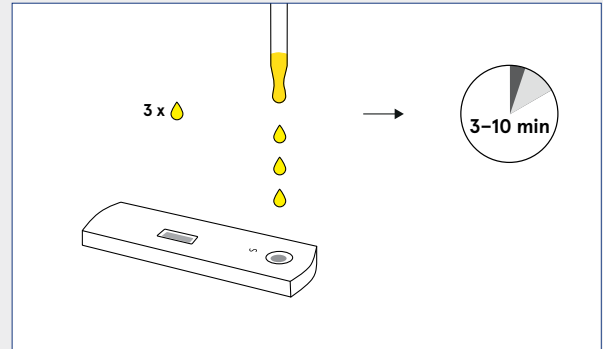

Place the test cassette on a clean and level surface. Hold the dropper vertically and add 3 drops of urine to the sample well. Start the timer.



Read the results at 3 minutes. Do not interpret the results after 10 minutes.

Interpretation of the results

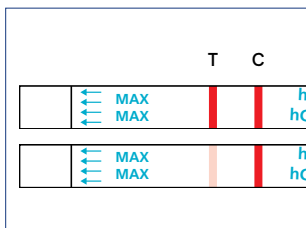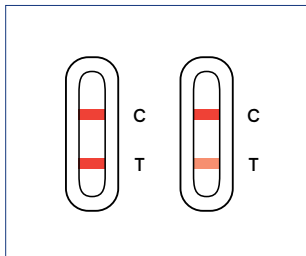



Positive result

Two coloured lines appear. One line appears in the control line region (C) and one line appears in the test line region (T). The test is qualitative and the quantitative value cannot be determined based on colour.

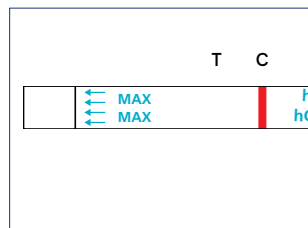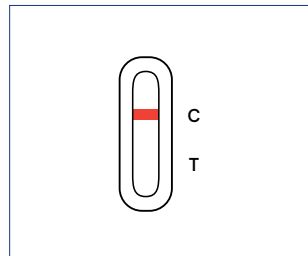

Negative result

One coloured line appears in the control line region (C). No line appears on the test line region (T).

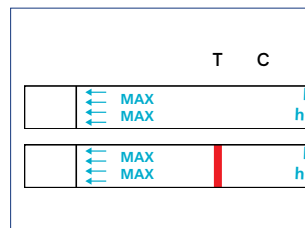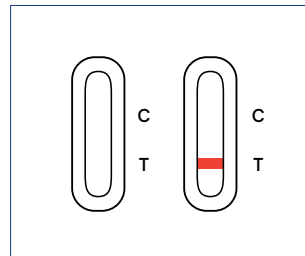

Invalid result

Control line (C) fails to appear. Review the procedure and repeat the test with a new test. If the problem persists, discontinue using the test kit and contact Aidian's representative.

Please consult the instructions for use before performing the test.

Alltest hCG Pregnancy Rapid tests

For professional *in vitro* diagnostic use only

Precautions, specimen collection and handling

- The tests should remain in sealed pouch or closed canister until use.
- The tests are stable for max. 90 days after first opening of a canister.
- First morning urine is preferred since it usually contains the highest hCG concentration.
- Collect the urine sample in a clean and dry container.
- Urine specimens may be stored at 2–8°C for up to 48 hours prior to testing. Samples containing visible precipitates should be centrifuged, filtered or allowed to settle to obtain a clear sample for testing. As with all diagnostic tests a confirmed pregnancy diagnosis should only be made by a physician after all needed clinical and laboratory findings have been evaluated.



Rapid test for early detection of hCG in urine

- Human Chorionic Gonadotropin (hCG) is produced by placenta and its concentration increases quickly after fertilisation making it an excellent marker for early detection of pregnancy. Alltest hCG Pregnancy Rapid tests are rapid chromatographic immunoassays to aid in early detection of pregnancy.

Easy to use

- Easy test procedure
- Clear colour reactions for easy results' interpretation

Reliable and fast

- Results in 3 minutes
- Internal control ensures correct test function



Contact info

Aidian Oy

aidian@aidian.eu

+358 10 309 3000

For orders: orders@aidian.eu

www.aidian.eu

Product	Product no.	Order no.
Alltest hCG 25 mIU/ml Pregnancy Dipstick (urine) 50 tests	FCH-101A	155220
Alltest hCG 40 mIU/ml Pregnancy Dipstick (urine) 50 tests	FCH-101B	155221
Alltest hCG 500 mIU/ml Pregnancy Dipstick (urine) 50 tests	FCH-101C	155222
Alltest hCG 25 mIU/ml Pregnancy Cassette (urine) 25 tests	FCH-102A	155223
Alltest hCG 40 mIU/ml Pregnancy Cassette (urine) 25 tests	FCH-102B	155224
Alltest hCG 500 mIU/ml Pregnancy Cassette (urine) 25 tests	FCH-102C	155225