

QuikRead go Plus Instrument software 2.4.1 release notes

A new QuikRead go Plus Instrument software version 2.4.1 is released on 28.5.2026.

The new software version has been developed and validated in compliance with IEC 62304:2006+AMD1:2015 Medical device software - Software life cycle processes and General Principles of Software Validation; Final Guidance for Industry and FDA Staff, Jan 11, 2002.

Accessory, workflow and usability improvements

- Added support for Dymo LabelWriter 400 and 450 printers.
- Barcode readers automatically move focus to the next input field in the measurement screen.
- Login workflow was updated to show the password field only after Operator ID has been input.
- Additional ID is now visible on A4 printed result list and exported file.
- Other user interface usability and translation improvements.

Connectivity and security improvements

- Improved LIS and POCT connectivity and WLAN stability.
- WLAN password maximum length increased to 100 characters.
- Added display of instrument WLAN MAC address in the About page.
- Security updates.

Availability of new software

After release, the new software version 2.4.1 is available on a per customer basis from www.softwareupdate.quikread.com and is available to order on USB stick (Cat. No 155433).