




QuikRead go HbA1c

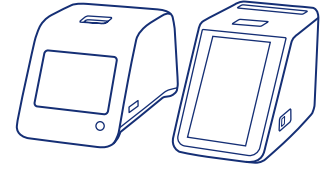


For QuikRead go Instrument and QuikRead go Plus Instrument

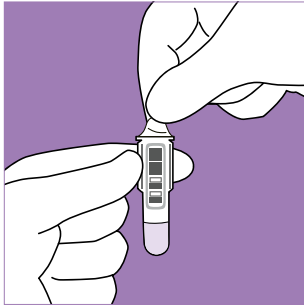


1 Remove the foil cover from the cuvette. Do not touch the clear flat surfaces in the lower part of the cuvette.

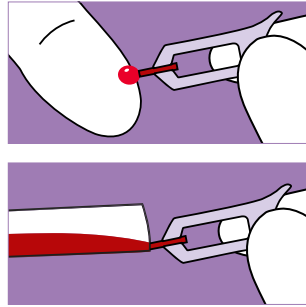


2 Hold QuikRead go Sample Collector 1µl almost horizontally just beneath the surface of the sample. Fill the capillary part of the sample collector completely.

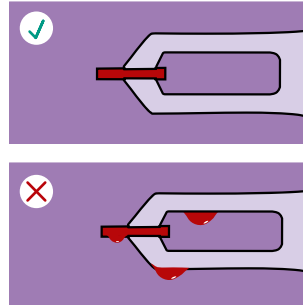


Note! Check that the capillary part is completely filled. Do not wipe the sample collector! If there is excess sample on the sample collector, discard the sample collector.

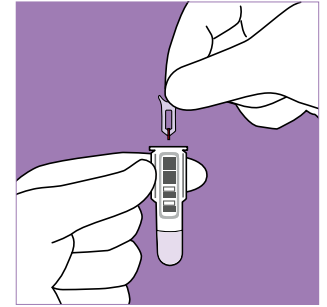


3 Insert the sample collector into the cuvette within 1 minute. The sample collector does not touch the solution in the cuvette.

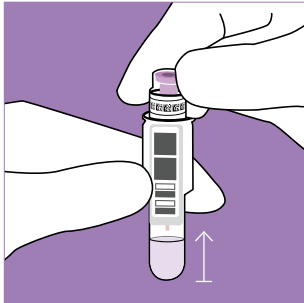

4 Close the cuvette tightly with the HbA1c reagent cap. Do not press down the violet inner part of the reagent cap. Perform the test within 5 minutes. Keep the cuvette upright and do not shake.

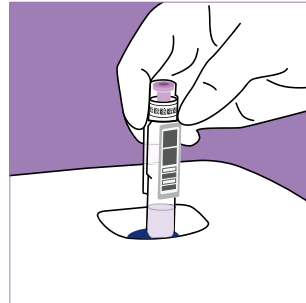


5 Choose Measure / Measure sample on the screen of the QuikRead go instrument. Insert the cuvette into the measurement well of the instrument. The barcode should be facing you.

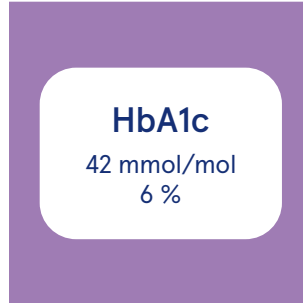

6 Measurement time is less than 6 minutes. The result is displayed on the screen, and the cuvette rises automatically when the measurement is completed.

Please consult the instructions for use before performing the test.

QuikRead go HbA1c

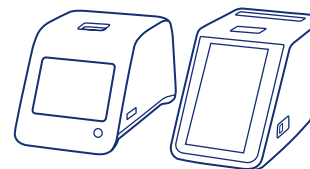

Kit

Unopened kit at 2–8°C: until the expiry date
Unopened kit at 18–25°C: 2 months



Reagent cap tube

Unopened tube at 2–8°C: until the expiry date of the kit
After the first opening at 2–8°C: 6 months / at 18–25°C: 2 months



Cuvettes

Unopened foil pouch at 2–8°C: until the expiry date of the kit
Unopened cuvettes after opening the foil pouch:
at 2–8°C: 6 months / at 18–25°C: 2 months



Opened cuvette

Let the buffer warm up to 18–25°C before testing
Perform the test within 1 hour from opening the cuvette
The sample is stable in the cuvette for 5 minutes with the closed cap



Sample collectors

Storage at 2–25°C: until the expiry date of the Sample Collector 1µl box



HbA1c Control Set

Unopened vial at 2–8°C: until the expiry date
Opened vial at 2–8°C: up to 50 days

Contact and orders

Aidian Oy, Finland

aidian@aidian.eu / +358 10 309 3000

orders@aidian.eu

quikread.com

Product	Cat. no.
QuikRead go HbA1c with QuikRead go Sample Collector 1µl, 25 tests	151058
QuikRead go HbA1c Control Set	154520
QuikRead go Sample Collector 1µl, 25 pcs	154457
QuikRead go Instrument	133893
QuikRead go Plus Instrument	155378