

QuikRead go<sup>®</sup> software changes

Following changes will take place when you update the QuikRead go software version

From 8.2.2 to 8.3.0	
Change	
Hardware driver update	
Localization text corrections	» Chinese language text corrections » Greek language text corrections
Wireless connection improvements	» Special character support for wireless SSID » WPA-EAP network support added
Result info screen correction	» Fixed instances where Buf and Reag lots have overlapped

From 8.1.1 to 8.2.2	
Change	
Modifications in user interface	» Fixed overlapping texts in iFOBTc
Support for new accessories	» GoDEX DT4x printer » TL-WN725N (Ver:3.0) WLAN adapter
Changes in reagents expire	» Allows measurements on the date when the reagents expire
Changes in connectivity features	» Improved POCT1-A2 logging for customers support and R&D purposes

From 7.6.2 to 8.1.1	
Change	

Aidian Marketing

Modifications in user interface	<div>» Possibility to disable emergency button from login screen is added</div> <div>» Password can be reset even when login is not used</div> <div>» GPrinter 5890 XIII paper roll printer support added</div> <div>» Company name change</div>
Support for new analytes	<div>» QuikRead go HbA1c can be measured</div>

From 7.5.1 to 7.6.2	
Change	
Changes in the instrument's internal operations	<div>» More quiet mixing sound with QuikRead go wrCRP</div>
Support for new analytes	<div>» QuikRead go easy CRP can be measured.</div>

Aidian Marketing

From 7.2.5 to 7.6.2

Change

New localisations	» New language (LV).
Modifications in user interface	<ul style="list-style-type: none"> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow/Maintenance.</li> </ul>
Changes in connectivity features	» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.
Support for new analytes	» QuikRead go easy CRP can be measured.

From 7.1.10 to 7.6.2

Change

New localisations	» New language (LV).
Modifications in user interface	<ul style="list-style-type: none"> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow/Maintenance.</li> </ul>
Changes in connectivity features	<ul style="list-style-type: none"> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>
Support for new analytes	» QuikRead go easy CRP can be measured.

## Aidian Marketing

From 6.5.2 to 7.6.2	
Change	
New localisations	» New language (LV).
Modifications in user interface	» Possible to set QC limits » Kit lot and expiration date shown during measurement » Number of Operator ID's stored in memory increased to eight » Animation showing how to correctly insert a cuvette » Unit for Hb in quality controls selectable from a list » New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot. » New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added. » Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite. » Lid closing time & Full standby time can be set down to 1 minute. » Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow/Maintenance.
Changes in connectivity features	» Bidirectional communication based on POCT1-A2. » WLAN support for LIS connections. » Results can be resent to LIS » Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments. » QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot. » Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.
Support for new analytes	» QuikRead go wrCRP and wrCRP+Hb products can be measured. » QuikRead go easy CRP can be measured.
Automation	» Support for two QuikRead go Instruments added on QuikRead go Feeder.

Aidian Marketing

From 6.3.4 to 7.6.2	
Change	
New localisations	» New languages (CN, ET, LT, LV).
Modifications in user interface	<ul style="list-style-type: none"> <li>» Power save option can be set from the settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow.</li> </ul>
Changes in connectivity features	<ul style="list-style-type: none"> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>
Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	» Support for two QuikRead go Instruments added on QuikRead go Feeder.

Aidian Marketing

From 6.1.5 to 7.6.2	
Change	
New localisations	» New languages (CN, ET, LT, LV).
Modifications in user interface	<ul style="list-style-type: none"> <li>» Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.</li> <li>» New About screen in the Maintenance settings.</li> <li>» Power save option can be set from the settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance.</li> </ul>
Changes in connectivity features	<ul style="list-style-type: none"> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» New LIS-feature settings in the ADMIN settings section.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>
Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» Label reading of the cuvette has been changed to be more robust.</li> <li>» The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.</li> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	» Support for two QuikRead go Instruments added on QuikRead go Feeder.

Aidian Marketing

From 5.1.7 to 7.6.2	
Change	
New localisations	» New languages (CN, ET, LT, LV).
Modifications in user interface	<ul style="list-style-type: none"> <li>» Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.</li> <li>» New About screen in the Maintenance settings.</li> <li>» Power save option can be set from the settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow.</li> </ul>
Changes in connectivity features	<ul style="list-style-type: none"> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» New LIS-feature settings in the ADMIN settings section.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>
Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» Label reading of the cuvette has been changed to be more robust.</li> <li>» The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.</li> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go iFOBT product can be measured.</li> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	» Support for two QuikRead go Instruments added on QuikRead go Feeder.

## Aidian Marketing

From 4.1.2 / 4.2.2 to 7.6.2	
Change	
New localisations	» New languages (CN, ET, LT, LV).
Modifications in user interface	<ul style="list-style-type: none"> <li>» Supports mmol/l unit for Hb measuring.</li> <li>» The Patient ID is shown on the display when the cuvette is asked to be inserted to start the measurement.</li> <li>» Extended memory capacity for 100 patient results and for 100 QC-results.</li> <li>» Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.</li> <li>» New About screen in the Maintenance settings.</li> <li>» Power save option can be set from the settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow/Maintenance.</li> </ul>
Changes in connectivity features	<ul style="list-style-type: none"> <li>» TCP/IP connectivity support.</li> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> <li>» New LIS-feature settings in the ADMIN settings section.</li> </ul>
Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» More precise error handling in case of temperature measurement.</li> <li>» Instrument's internal cuvette position check boundaries and cuvette's positioning have been adjusted for a more robust detection of the cuvette.</li> <li>» Label reading of the cuvette has been changed to be more robust.</li> <li>» The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.</li> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go hsCRP+Hb product can be measured.</li> <li>» QuikRead go iFOBT product can be measured.</li> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>



Aidian Marketing

Automation	» Support for two QuikRead go Instruments added on QuikRead go Feeder.
------------	--

From 3.1.2 / 3.1.3 / 3.1.6 to 7.6.2	
Change	
New localisations	» New languages (CN, ET, LT, LV).
Modifications in user interface	<ul style="list-style-type: none"> <li>» Result information divided into two separate displays.</li> <li>» Supports mmol/l unit for Hb measuring.</li> <li>» The Patient ID is shown on the display when the cuvette is asked to be inserted to start the measurement.</li> <li>» Extended memory capacity for 100 patient results and for 100 QC-results.</li> <li>» Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.</li> <li>» New About screen in the Maintenance settings.</li> <li>» Power save option can be set from the settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow/Maintenance.</li> </ul>
Changes in connectivity features	<ul style="list-style-type: none"> <li>» TCP/IP connectivity support.</li> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» New LIS-feature settings in the ADMIN settings section.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>
Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» More precise error handling.</li> <li>» Instrument's internal cuvette position check boundaries and cuvette's positioning have been adjusted for a more robust detection of the cuvette.</li> <li>» Label reading of the cuvette has been changed to be more robust.</li> <li>» The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.</li> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>

Aidian Marketing

Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go CRP+Hb product can be measured.</li> <li>» QuikRead go Strep A product can be measured.</li> <li>» QuikRead go hsCRP+Hb product can be measured.</li> <li>» QuikRead go iFOBT product can be measured.</li> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	<ul style="list-style-type: none"> <li>» Support for two QuikRead go Instruments added on QuikRead go Feeder.</li> </ul>

From 2.4.13 to 7.6.2	
Change	
New localisations	<ul style="list-style-type: none"> <li>» New languages (CN, ET, LT, LV).</li> </ul>
Modifications in user interface	<ul style="list-style-type: none"> <li>» Result information divided into two separate displays.</li> <li>» Supports mmol/l unit for Hb measuring.</li> <li>» The Patient ID is shown on the display when the cuvette is asked to be inserted to start the measurement.</li> <li>» Extended memory capacity for 100 patient results and for 100 QC-results.</li> <li>» Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.</li> <li>» New About screen in the Maintenance settings.</li> <li>» Power save option can be set from the settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow/Maintenance.</li> </ul>
Changes in connectivity features	<ul style="list-style-type: none"> <li>» TCP/IP connectivity support.</li> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» New LIS-feature settings in the ADMIN settings section.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>

## Aidian Marketing

Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» More precise error handling.</li> <li>» Instrument's internal cuvette position check boundaries and cuvette's positioning have been adjusted for a more robust detection of the cuvette.</li> <li>» Label reading of the cuvette has been changed to be more robust.</li> <li>» The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.</li> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>
Change in the hematocrit range with QuikRead go CRP test	<ul style="list-style-type: none"> <li>» Low limit for hematocrit with whole blood sample is changed from 20% to 15%.</li> <li>» High limit for hematocrit with whole blood sample is changed from 60% to 75%.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go CRP+Hb product can be measured.</li> <li>» QuikRead go Strep A product can be measured.</li> <li>» QuikRead go hsCRP+Hb product can be measured.</li> <li>» QuikRead go iFOBT product can be measured.</li> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	<ul style="list-style-type: none"> <li>» Support for two QuikRead go Instruments added on QuikRead go Feeder.</li> </ul>

## From 2.4.11 to 7.6.2

### Change

New localisations	<ul style="list-style-type: none"> <li>» New languages (CN, ET, LT, LV).</li> </ul>
Modifications in user interface	<ul style="list-style-type: none"> <li>» Result information divided into two separate displays.</li> <li>» Supports mmol/l unit for Hb measuring.</li> <li>» The Patient ID is shown on the display when the cuvette is asked to be inserted to start the measurement.</li> <li>» Extended memory capacity for 100 patient results and for 100 QC-results.</li> <li>» Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.</li> <li>» New About screen in the Maintenance settings.</li> <li>» Power save option can be set from the settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Measurement flow/Maintenance.</li> </ul>

## Aidian Marketing

Changes in connectivity features	<ul style="list-style-type: none"> <li>» TCP/IP connectivity support.</li> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» New LIS-feature settings in the ADMIN settings section.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>
Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» More precise error handling.</li> <li>» Instrument's internal cuvette position check boundaries and cuvette's positioning have been adjusted for a more robust detection of the cuvette.</li> <li>» Label reading of the cuvette has been changed to be more robust.</li> <li>» The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.</li> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>
Changes in the display usage	<ul style="list-style-type: none"> <li>» Smoother display usage.</li> </ul>
Change in the hematocrit range with QuikRead go CRP test	<ul style="list-style-type: none"> <li>» Low limit for hematocrit with whole blood sample is changed from 20% to 15%.</li> <li>» High limit for hematocrit with whole blood sample is changed from 60% to 75%.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go CRP+Hb product can be measured.</li> <li>» QuikRead go Strep A product can be measured.</li> <li>» QuikRead go hsCRP+Hb product can be measured.</li> <li>» QuikRead go iFOBT product can be measured.</li> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	<ul style="list-style-type: none"> <li>» Support for two QuikRead go Instruments added on QuikRead go Feeder.</li> </ul>

## Aidian Marketing

## From 2.4.10 to 7.6.2

## Change

## New localisations

» New languages (CN, ET, LT, LV).

## Modifications in user interface

- » Result information divided into two separate displays.
- » Supports mmol/l unit for Hb measuring.
- » The Patient ID is shown on the display when the cuvette is asked to be inserted to start the measurement.
- » Extended memory capacity for 100 patient results and for 100 QC-results.
- » Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.
- » New About screen in the Maintenance settings.
- » Power save option can be set from the settings wizard.
- » Possible to set QC limits
- » Kit lot and expiration date shown during measurement
- » Number of Operator ID's stored in memory increased to eight
- » Animation showing how to correctly insert a cuvette
- » Unit for Hb in quality controls selectable from a list
- » New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.
- » New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.
- » Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.
- » Lid closing time & Full standby time can be set down to 1 minute.
- » Separate files for Error, Instrument, and Security logs in Settings/Measurement flow/Maintenance.

## Changes in connectivity features

- » Results can be resent to LIS
- » Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.
- » QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.
- » TCP/IP connectivity support.
- » New LIS-feature settings in the ADMIN settings section.
- » Bidirectional communication based on POCT1-A2.
- » WLAN support for LIS connections.
- » Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.

## Changes in the instrument's internal operations

- » More precise error handling.
- » Instrument's internal cuvette position check boundaries and cuvette's positioning have been adjusted for a more robust detection of the cuvette.
- » Label reading of the cuvette has been changed to be more robust.
- » The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.
- » DHCP address request retry amount increased to get IP address more reliably.
- » Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.

## Changes in the display usage

- » Smoother display usage.

## Aidian Marketing

Change in the hematocrit range with QuikRead go CRP test	<ul style="list-style-type: none"> <li>» Low limit for hematocrit with whole blood sample is changed from 20% to 15%.</li> <li>» High limit for hematocrit with whole blood sample is changed from 60% to 75%.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go CRP+Hb product can be measured.</li> <li>» QuikRead go Strep A product can be measured.</li> <li>» QuikRead go hsCRP+Hb product can be measured.</li> <li>» QuikRead go iFOBT product can be measured.</li> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	<ul style="list-style-type: none"> <li>» Support for two QuikRead go Instruments added on QuikRead go Feeder.</li> </ul>

## From 2.3.6 to 7.6.2


### Change


New localisations	<ul style="list-style-type: none"> <li>» New languages (SL, SR, HR, EL, FR, DE, IT, CZ, SK, HU, PL, ES, PT, NL, CN, ET, LT, LV).</li> <li>» Small changes to the Nordic languages.</li> </ul>
Modifications in user interface	<ul style="list-style-type: none"> <li>» Result information divided into two separate displays.</li> <li>» The instrument will give more precise information if the accu-unit is getting empty.</li> <li>» After the instrument is unused for a specified time it will go to a sleep mode.</li> <li>» The ADMIN password is asked before the test parameters can be changed.</li> <li>» It is no longer possible to make software rollback to the previous version.</li> <li>» Supports mmol/l unit for Hb measuring.</li> <li>» The Patient ID is shown on the display when the cuvette is asked to be inserted to start the measurement.</li> <li>» Extended memory capacity for 100 patient results and for 100 QC-results.</li> <li>» Changed power save mode settings. Hibernate functionalities Full standby or Close lid only.</li> <li>» New About screen in the Maintenance settings.</li> <li>» Powersave option can be set from the Settings wizard.</li> <li>» Possible to set QC limits</li> <li>» Kit lot and expiration date shown during measurement</li> <li>» Number of Operator ID's stored in memory increased to eight</li> <li>» Animation showing how to correctly insert a cuvette</li> <li>» Unit for Hb in quality controls selectable from a list</li> <li>» New QC lots can be added on site. QuikRead go must be shut down and restarted after adding a new QC lot.</li> <li>» New feature Security settings enable user authentication with operator ID and password. A session inactivity timer also added.</li> <li>» Admin (and supervisor if Security settings ON) can edit and delete QC lots onsite.</li> <li>» Lid closing time &amp; Full standby time can be set down to 1 minute.</li> <li>» Separate files for Error, Instrument, and Security logs in Settings/Maintenance flow/Maintenance.</li> </ul>

## Aidian Marketing

Changes in connectivity features	<ul style="list-style-type: none"> <li>» Results can be resent to LIS</li> <li>» Third Patient ID, e.g. birthdate or Doctor ID, can be added to patient results. Will be sent to LIS from POCT1-A2 connected instruments.</li> <li>» QC lots can be added manually on site. QuikRead go must be shut down and restarted after manually adding a new QC lot.</li> <li>» LIS offline data: if the LIS offline memory is full, the instrument informs about it.</li> <li>» TCP/IP connectivity support.</li> <li>» New LIS-features settings in the ADMIN settings section.</li> <li>» Bidirectional communication based on POCT1-A2.</li> <li>» WLAN support for LIS connections.</li> <li>» Local user management and POCT1-A2 can be simultaneously used except when POCT1-A2 user login is enabled.</li> </ul>
Changes in the instrument's internal operations	<ul style="list-style-type: none"> <li>» An automatic hall-sensor calibration during every start-up.</li> <li>» A change in the lift position during the transportation to secure that the lift position can not change during the transportation.</li> <li>» More precise error handling.</li> <li>» Instrument's internal cuvette position check boundaries and cuvette's positioning have been adjusted for a more robust detection of the cuvette.</li> <li>» Label reading of the cuvette has been changed to be more robust.</li> <li>» The instrument checks before the measurement that the cap is placed properly. In case the cap is left too high, the instrument informs Please check the cap.</li> <li>» DHCP address request retry amount increased to get IP address more reliably.</li> <li>» Optimized maintenance logging feature to avoid occasional slowing down of the instrument at 97% progress.</li> </ul>
Changes in the display usage	<ul style="list-style-type: none"> <li>» Smoother display usage.</li> </ul>
Change in the hematocrit range with QuikRead go CRP test	<ul style="list-style-type: none"> <li>» Low limit for hematocrit with whole blood sample is changed from 20% to 15%.</li> <li>» High limit for hematocrit with whole blood sample is changed from 60% to 75%.</li> </ul>
Support for new analytes	<ul style="list-style-type: none"> <li>» QuikRead go CRP+Hb product can be measured.</li> <li>» QuikRead go Strep A product can be measured.</li> <li>» QuikRead go hsCRP+Hb product can be measured.</li> <li>» QuikRead go iFOBT product can be measured.</li> <li>» QuikRead go wrCRP and wrCRP+Hb products can be measured.</li> <li>» QuikRead go easy CRP can be measured.</li> </ul>
Automation	<ul style="list-style-type: none"> <li>» Support for two QuikRead go Instruments added on QuikRead go Feeder.</li> </ul>

Signatures:

<b>Document Updates Review:</b>	I have reviewed the updated contents of this document.
Name:	Kivirauma, Tapio
2025-01-31 13:16:39 (UTC+00:00)	
Electronically Signed in	Timestamp
	

<b>Document Updates Review:</b>	I have reviewed the updated contents of this document.
Name:	Dahal, Suman
2025-02-05 10:38:09 (UTC+00:00)	
Electronically Signed in	Timestamp
	

<b>Document Updates Approval:</b>	I have Approved the updated content of this document and found no errors
Name:	Kuutti, Mirjami
2025-02-07 07:59:24 (UTC+00:00)	
Electronically Signed in	Timestamp
