



# BIOSYNEX COMBO FLU-COVID BSS

## VIROLOGY

Distributie Nederland:



info@mediphos.nl  
www.mediphos.com

Distribution Belgium - Luxembourg:



info@mediphos.be

Industrieweg 12-B, 6871 KA Renkum (NL), tel: +31 317 351838



*A **rapid test** for the qualitative and differential detection of **Influenza A & B and SARS-CoV-2 antigens** from a single nasopharyngeal swab*



*One test to determine which virus is responsible for the symptoms in order to adapt patient management*

- Influenza A & B and/or SARS-CoV-2



*Fast and easy to handle*

- Results in 15 minutes only
- From a single nasopharyngeal sample



*Reliable and proven performances*

- Evaluated on more than 200 clinical samples for each pathogen\*
- Highly sensitive and specific






# BIOSYNEX COMBO FLU-COVID BSS

As COVID-19 and Flu symptoms are similar (fever, dry cough, fatigue, runny or stuffy nose), it is essential that the pathogen responsible for

the symptoms be rapidly identified in order to take the appropriate therapeutic or even isolation measures necessary to control the pandemics.

**BIOSYNEX COMBO FLU-COVID BSS** allows the qualitative and differential detection of antigens from 3 different viruses : Influenza A (including H1N1), Influenza B and SARS-CoV-2.

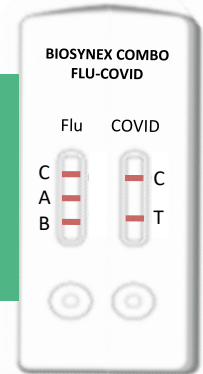

## PROCEDURE#

#Please refer to Package insert for full procedure

- Add 13-17 drops of the extraction buffer.
- Carefully collect the sample on the posterior nasopharynx.
- Discharge the swab by rotations and leave to soak for 1 minute.
- Place 4 drops of the solution in each S well.
- Read at 15 minutes. Do not interpret after 20 minutes.

## PERFORMANCES\*

\*Please refer to Package insert for complete clinical evaluations and performances

	Influenza A	Influenza B	COVID-19
Sensitivity	93%	90%	96%
Specificity	96%	96%	100%



For standardised and automated interpretation, the BSX Reader is available

## KIT CONTENTS

### BIOSYNEX COMBO FLU-COVID BSS

- 25 test cassettes
- 25 sterile nasopharyngeal swabs (CE0197)
- 25 extraction tubes
- 25 dropper-caps
- 2 vials of extraction buffer
- 1 workstation
- Package insert

Ref. SW40008

24 months

2-30°C

Set of + & - controls available separately SW40009



BIOSYNEX SA  
22 Boulevard Sébastien Brant  
67400 ILLKIRCH-GRAFFENSTADEN - France  
Tel. : +33 3 88 77 57 52  
Fax. : +33 3 88 78 76 78  
export@biosynex.com  
www.biosynex.com

Manufacturer BIOSYNEX Swiss SA  
(Switzerland)

*In vitro* Diagnostics.  
Read carefully the instructions for use  
on the package insert.